



AXIUS™ 8mm Needle Driver

QUICK REFERENCE CLEANING GUIDE



Scan for IFU



1 CHECK LIFE

If reprocessed **10x**,
dispose of expired device



2 PRIME

Prime top flush port with
20ml cleaning solution
while articulating **3x**
Ensure jaws are open



3 SOAK DEVICE

30 min.



4 FLUSH PORTS

Until water runs clear

2x 20 sec.



5 SPRAY TIP

While device is submerged
& articulating the tip

30 sec.



6 BRUSH

Tip & shaft for 1 min.

Frame & handle for 5 min.

1 min.
5 min.



7 RINSE DEVICE

Until all visible soil
& cleaning agents removed

1 min.



8 INSPECT

If soiled, repeat cleaning



CONSULT OPERATING INSTRUCTIONS. For use with pH-neutral cleaning solution only.
Note: This is a process step guide and not intended to replace the Instructions for Use.
All detailed steps, warnings, and precautions are provided in the Instructions for Use.

Property of FlexDex Surgical P/N: 603004 Rev B (2024-07-26)

FlexDex
SURGICAL