



AXIUS™ 8mm Needle Driver

QUICK REFERENCE STERILIZATION GUIDE



Scan for IFU

1



CHECK LIFE

If reprocessed **10x**,
dispose of expired device

2



DRY

Drain water and dry device
with low-linting wipe

3



INSPECT


Inspect for soil, device
damage & flush tube position

4



LUBRICATE

Apply 1-2 drops on cable
location, excluding jaws

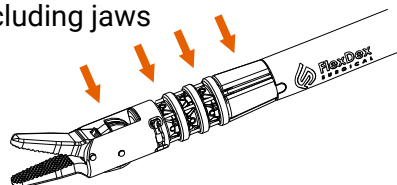



5



PACK & WRAP TRAY

Prior to wrapping, check:

- Handle unlatched & jaws open
- Flush ports open (caps detached)
- Attach shaft clamp securely to device with gimbals seated correctly

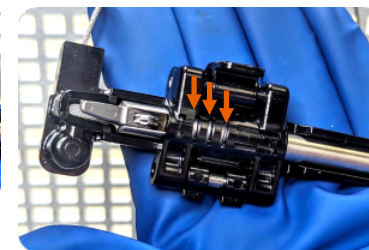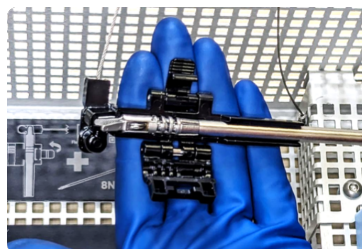


6



STERILIZE & TRACK

Sterilize device per IFU
& update hospital tracking

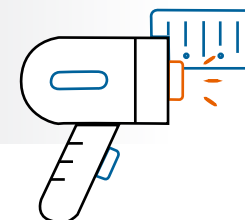

CONSULT OPERATING INSTRUCTIONS.

Note: This is a process step guide and not intended to replace the Instructions for Use. All detailed steps, warnings, and precautions are provided in the Instructions for Use.

Property of FlexDex Surgical P/N: 603005 Rev B (2024-08-06)

FlexDex
SURGICAL®