



# AXIUS™ 8mm Needle Driver

## QUICK REFERENCE OPERATING ROOM GUIDE



Scan for IFU

### PRIOR TO USE: FUNCTIONALITY CHECK

1

**CLOSE FLUSH PORTS**



3

**ROTATE DIAL**

Dial should rotate freely



2

**ARTICULATE HANDLE**

Tip should smoothly follow handle in all directions





4

**LOCK / UNLOCK LEVER**



### AFTER USE: PACK IN TRAY

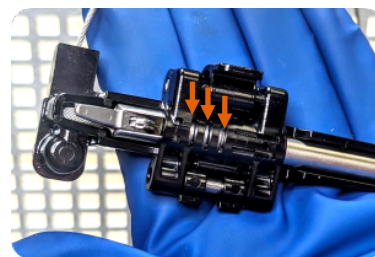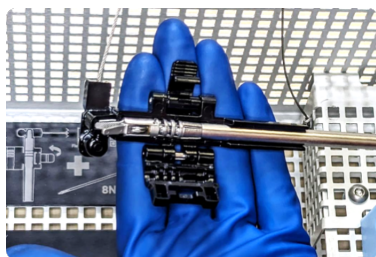

- Attach shaft clamp securely to device with gimbals seated correctly
- If applicable, track the device within the device management system
- Send for reprocessing

**CONSULT OPERATING INSTRUCTIONS. Note:** This is a process step guide and not intended to replace the Instructions for Use. All detailed steps, warnings, and precautions are provided in the Instructions for Use.

**Contraindications:** Do **NOT** use the AXIUS 8mm Needle Driver as a grasper or dissector, in procedures within the central nervous / circulatory systems, or where minimally invasive procedures are contraindicated.

Property of FlexDex Surgical P/N: 603006 Rev B (2024-08-06)

**FlexDex**  
SURGICAL®